Gebrauchsanweisung  
 Instructions for use  
 Mode d'emploi  
 Istruzioni per l'uso  
 Instrucciones de empleo

## GCER Universal Bonding Kit

Manufacturer | Hersteller  
 Distribution | Vertrieb

**Amann Girrbach AG**  
 Herrschaftswiesen 1  
 6842 Koblach | Austria  
 Fon +43 5523 62333-105  
 Fax +43 5523 62333-5119  
 austria@amanngirrbach.com  
 www.amanngirrbach.com

Made in the European Union

www.hilblbau.com 3228FB 2012-07-02

**AMANNGIRRBACH**

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Original-Gebrauchsanweisung

### PRODUKTBESCHREIBUNG

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## INSTRUCTIONS FOR USE

Translation of the original instructions for use

### DESCRIPTION OF THE PRODUCT

Bonding aid for bonding glass-ceramic blanks and lithium disilicate blanks using the Amann Girrbach holder, includes 2 bonding holders for the different sizes of blank (10 x 12 x 15 mm, 12 x 14 x 18 mm)

### APPLICATION

- Grip the glass-ceramic holder (4) in the mounting block (1). The groove in the holder should point forwards.
- Apply a thin coat of bonder (recommendation: Loctite 454) to the entire bonding surface of the holder (4a).
- Place the appropriate bonding holder (2) in position:  
 \_ White for size 10 x 12 x 15 blanks  
 \_ Black for size 12 x 14 x 18 blanks
- Then use the bonding holder to position the glass-ceramic blank and press it into place; the blank should be seated in the middle of the bonding aid (3). Caution: press the blank directly into place and ensure that it does not move!
- When bonding glass-ceramic blanks with different shade layers, the lightest layer must point towards the screw side of the mounting block; when grinding in a Ceramill Motion 2, this is the occlusal side.

### PLEASE NOTE / IMPORTANT HINTS FOR USE

- The bonding surfaces of the glass-ceramic holder and glass-ceramic blank must be absolutely grease-free and dry (steam-clean the surfaces or clean them with alcohol).
- It is not advisable to sandblast the bonding surfaces. Sandblasting does not enhance bond strength and may, possibly, damage the blank.
- After bonding, the bonded joint must be allowed to harden for 30 mins. before grinding the glass-ceramic in a Ceramill Motion 2.
- The contact areas must be wetted fully with bonder; an inadequate amount of bonder may cause the blank to loosen during grinding.
- After bonding, the blank should not be exposed to excessive temperature fluctuations.
- Blanks with a thicker side (ridge) must be placed in the bonding aid with the "ridge" facing towards the holder.
- It is advisable to use a new holder for each new blank. Should this not be the case, ensure that no bonder remains on the bonding surfaces.

### PRESENTATION

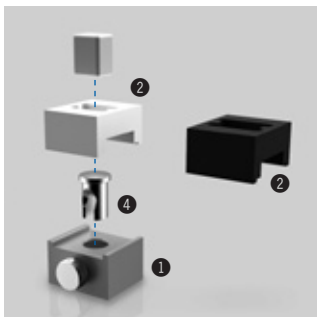
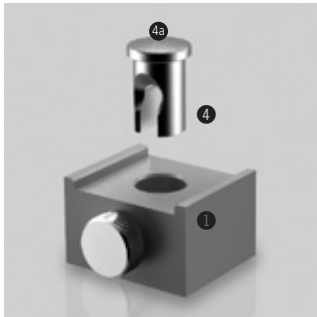
760973 GCER Universal Bonding Kit

### ACCESSORY

760974 Universal holder for  
 glass-ceramic blanks (10 pcs.)

### GUARANTEE

Application technological recommendations, regardless of whether they are given orally, in writing or during practical training, are based on our own experience and tests and must, therefore, be considered guidelines only. Our products are subject to continuous further development. We therefore reserve the right to modify the design and composition.

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