
ceramill motion 2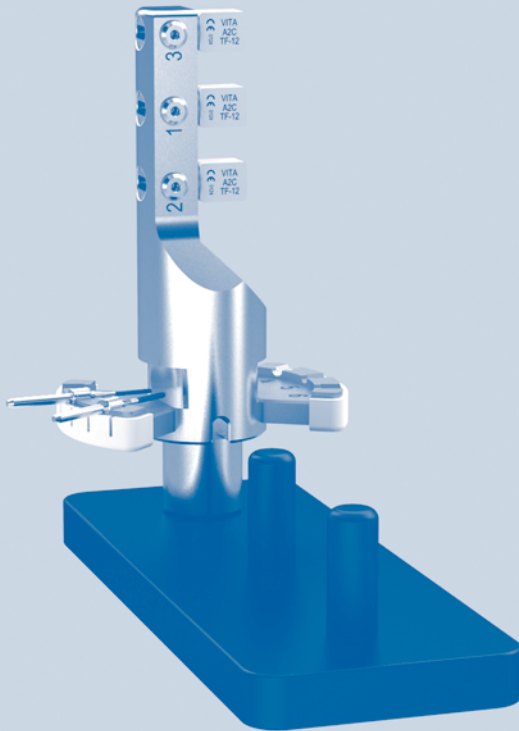

Gebrauchsanweisung
Instructions for use
Instruction Manual
Instructions
d'utilisation
Istruzioni per l'uso
Instrucciones de uso

Deutsch	2 - 5
English	6 - 9
Français	10 - 13
Italiano	14 - 18
Español	19 - 22

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3 Product Description 6

4 Application 7

4.1 Loading the Blank Holder for Glass Ceramic Blanks 7

4.2 Preparing the Ceramill Motion 2 ... 8

4.3 Inserting the Blank Holder for Glass Ceramic Blanks 9

4.4 Starting the Grinding Program 9

5 Warranty/Exclusion of Liability 9

6 Download Infos 9

1 Explanation of Symbols

Warning indications

Warning indications in the text are marked with a colour-backed triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that serious personal injury can occur.

Important information

Important information that do not lead to hazards for humans or property damage are marked with the icon aside and are also boxed.

Other symbols in the Manual

Symbol	Meaning
>	Item of an operation description
—	Item of a list
▪	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Tab. 1

2 Suitable Personnel

NOTICE:

The product may only be used by trained specialised personnel.

3 Product Description

The blank holder for glass ceramic blanks (179260) is a blank holder for Ceramill Motion 2. It is used to accommodate up to three glass ceramic blanks and the respective Ceramill Motion 2 Diamond tools.

Delivery Scope

- _ Blank holder for glass ceramic blanks (3-fold)
- _ Storage rack

4 Application

4.1 Loading the Blank Holder for Glass Ceramic Blanks

The blank holder can also be inserted into the Ceramill Motion 2 before loading it (see page 5).

▷ Insert the Ceramill Motion 2 Diamond standard tools into the blank holder for glass ceramic blanks. Position the tool shank facing toward the outside.

- Mount the Ceramill Motion 2 Diamond 1.8 (760622) at position 1.
- Mount the Ceramill Motion 2 Diamond 1.4 (760621) at position 2.

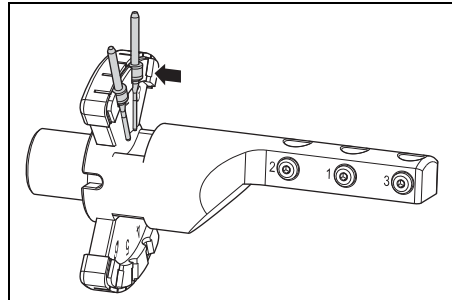
Fig. 1

▷ Optional: Insert the Ceramill Motion 2 Diamond sister tools into the blank holder for glass ceramic blanks. Position the tool shank facing toward the outside.

- Mount the Ceramill Motion 2 Diamond 1.8 (760622) at position 4.
- Mount the Ceramill Motion 2 Diamond 1.4 (760621) at position 5.

▷ Depending on your selection in the Ceramill Match 2, insert up to three blanks into the blank holder with the groove facing frontward, and secure them with the Allen key.

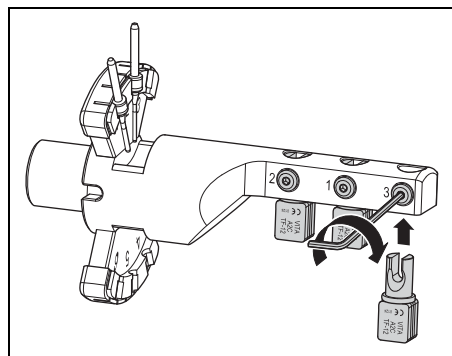
Fig. 2

4.2 Preparing the Ceramill Motion 2

When the Ceramill Motion 2 is set up for dry processing:

- ▷ Convert the Ceramill Motion 2 for wet processing (see Ceramill Motion 2 Operating Instructions)
- ▷ Disengage lever [3] and pull off the transparent splash guard [1] toward the front.

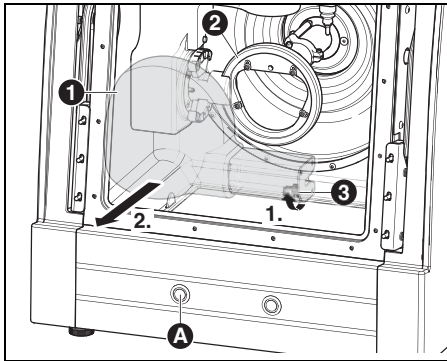
Fig. 3

- 1 Splash guard
- 2 Blank holder
- 3 Lever
- A Operation button

To remove the blank holder:

- ▷ With the door closed, press operation button [A] for at least three seconds. The current tool is placed down and the blank holder [2] returns to the change position.

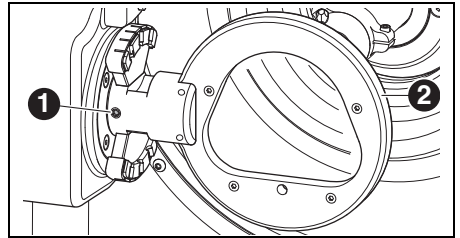
Fig. 4

- 1 Fastening screw for blank holder
- 2 Blank holder
- ▷ Loosen fastening screw [1].
- ▷ Pull off the blank holder toward the right.
- ▷ Clean the locating bore and surfaces.

Blank holders not in use can be placed on the storage rack and thus securely stored.

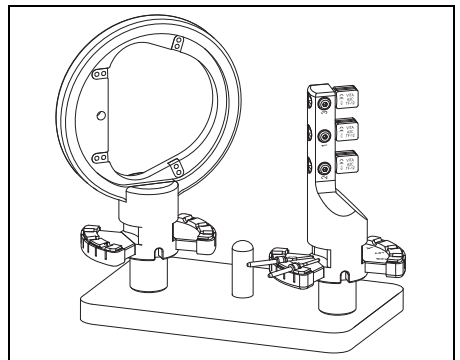
Fig. 5

4.3 Inserting the Blank Holder for Glass Ceramic Blanks

- ▷ Check the pin of the blank holder and clean if required.
- ▷ Insert the blank holder with the pin **to the stop** in the locating bore.

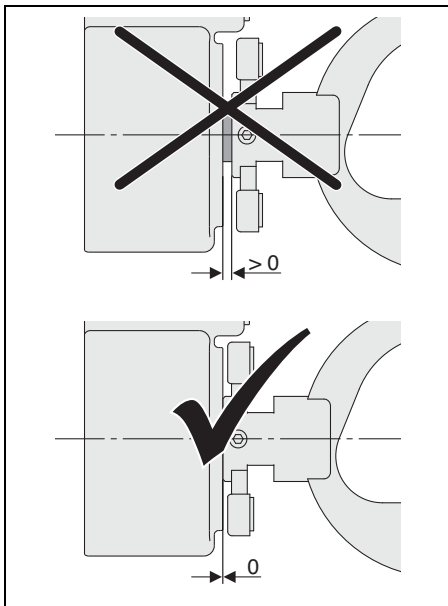
Fig. 6

NOTICE:

Inaccurate working results or damage to the blank-holder fixture possible!

- ▷ Tighten fastening screw [1] with a torque of 4 Nm.

The torque wrench included in the delivery scope of the Ceramill Motion 2 is appropriately set.

- ▷ Tighten fastening screw [1] with with the torque wrench.

4.4 Starting the Grinding Program

- ▷ Briefly press operation button [A].
The machine advances to the starting position.
- ▷ Mount the transparent splash guard [1] to the extraction opening and lock it in place with lever [3].
- ▷ Transfer the milling file for wet processing.

5 Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development. Subject to changes in handling and composition.

6 Download Infos

Further instructions/manuals and video tutorials can be downloaded under www.ceramill-m-center.com and/or www.amanngirrbach.com.

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Amann Gırrbach AG
Amann Gırrbach GmbH